



Quick Guide

Rev 1.8

Please refer to the Instructions for Use
for detailed information

Operatory Use

1. After handpiece use, remove the rotary instrument from the handpiece using safe methods and discard into an appropriate sharps container.
2. Remove the handpiece from the coupling or dental tubing.
3. Open the RiteWipe pouch dispenser cap and seal to remove a wipe.
4. Close the cap and seal on the RiteWipe pouch dispenser to assure wipe integrity and cleanliness.
5. With the RiteWipe “Blue Dot Technology” facing towards the handpiece surface, wipe the handpiece in a rotational direction around the handpiece, touching all surfaces, starting at the head and moving towards the bottom. Reference: [Page 21, CDC Guidelines – 2003](#).
6. Rotate the handpiece within the wipe several times until debris and bioburden are no longer visible, paying close attention to the fiber optic lens area (if present), water port area, and uneven surfaces.
7. After use, discard the RiteWipe into an appropriate waste container. Reference section 5 of long form IFU for detailed information.
8. Ready for transport to the Central Processing Area.



Quick Guide

Rev 1.8

Please refer to the Instructions for Use for detailed information

Central Processing Area

1. Remove the handpiece from the transportation container using safe methods.
2. Open the RiteWipe pouch dispenser cap and seal to remove a wipe.
3. Close the cap and seal on the RiteWipe pouch dispenser to assure wipe integrity and cleanliness.
4. Scrub the handpiece under running water with a RiteWipe in a horizontal back and forth motion, rotating the handpiece to clean the entire surface, including the head. Ensure all visible debris and bioburden is removed. [Reference: Page 21, CDC Guidelines – 2003](#)
5. After scrubbing, continue to rinse the handpiece under running water to remove any excess residual. [Reference: Page 21, CDC Guidelines – 2003](#)
6. After use, discard the RiteWipe into an appropriate waste container. Reference section 5 of long form IFU for detailed information.
7. Dry the handpiece with a lint-free towel in preparation for your handpiece manufacturer's internal maintenance process (Refer to your handpiece manufacturer's maintenance and sterilization process and/or IFU). Reference: [Page 15, CDC Summary – 2016](#).

[Morbidity and Mortality Weekly Report, Recommendations and Reports, December 19, 2003 / Vol. 52 / No. RR-17: Guidelines for Infection Control in Dental Health-Care Settings – 2003; DEPARTMENT OF HEALTH AND HUMAN SERVICES CENTERS FOR DISEASE CONTROL AND PREVENTION](#)

[Centers for Disease Control and Prevention, Summary of Infection Prevention Practices in Dental Settings: Basic Expectations for Safe Care. Atlanta, GA: US Department of Health and Human Services, Centers for Disease Control and Prevention, National Center for Chronic Disease Prevention and Health Promotion, Division of Oral Health; March 2016.](#)